

# Pairing MemorialCare Pulse Oximeter

Only pair your MemorialCare Pulse Oximeter after you've successfully connected your MemorialCare Virtual Exam Kit. You will only need to perform this setup once.

**Step 1:** Turn on the MemorialCare Virtual Exam Kit by pressing on the power button located on top of the device and wait for it to connect to a network.



**Step 2:** Insert two AA batteries into your Pulse Oximeter.



**Step 3:** Connect the smaller end of the adapter cable into the Pulse Oximeter.



**Step 4:** Connect the bigger end of the adapter cable to the back of the Virtual Exam Kit and turn on the Pulse Oximeter.



**Step 5:** A message on the Virtual Exam Kit will display indicating the setup is complete.