

Pairing TytoCare™ Pulse Oximeter

Only pair your TytoCare Pulse Oximeter after you've successfully connected your TytoCare device. You will only need to perform this setup once.

Step 1: Turn on the TytoCare device by pressing on the power button located on top of the device and wait for it to connect to a network.



Step 2: Insert 2 AA batteries into your Pulse Oximeter.



Step 3: Connect the smaller end of the adapter cable into the Pulse Oximeter.



Step 4: Connect the bigger end of the adapter cable to the back of the TytoCare device and turn on the Pulse Oximeter.



Step 5: A message on the TytoCare device will display indicating the setup is complete.